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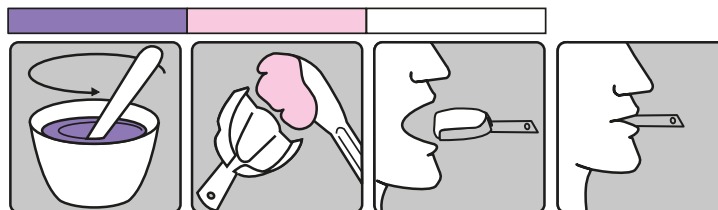
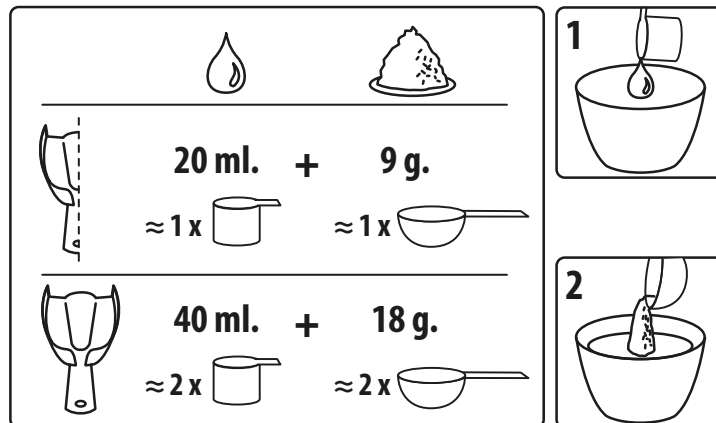
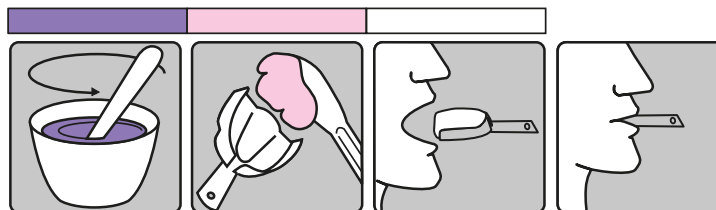
(*) Measurements taken at 23°C with deionised water.

(*) mixing time 45"

(*) working time 2'45"

(*) initial setting time 2'55"

only 30" in mouth

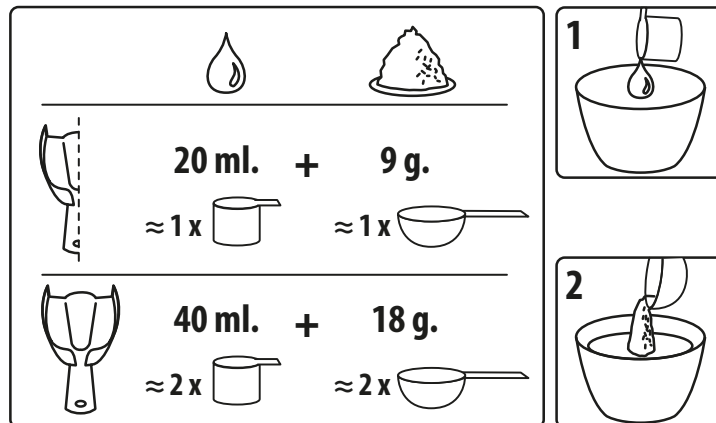
It does not require special treatment before use. Follow the powder/water ratio. Pour the powder into the water, mix vigorously. Work in compliance with the declared times (*), measurements were taken at 23°C with deionized water. When using specific mechanical mixers for alginate-based impression materials, follow the manufacturer's instructions. Variations in powder/water ratio, in temperature and/or water hardness and in the speed and/or friction during mixing cause changes in the working time and in the initial setting time. **The colour changes indicate the different work phases (see table).** Wash, rinse, shake and disinfect the impression (specific alcohol-based spray disinfectants are recommended). By keeping the impression in a sealed plastic bag, without adding anything else, the model can be poured within 168 hours.

Recommended plasters: Kromotipo 3, Singletipo 3, Kromotipo 4, Singletipo 4, Ortotipo.


WARNINGS: For use, wear face mask, protective gloves, eye protection and protective clothing. The powder is "Dust Free", however, caution is recommended to avoid inhalation. In case of contact with eyes wash thoroughly with water. After taking the powder close the bag immediately. Store in a cool and dry place. Use within the expiry date. Do not use on patients allergic to mint flavour. Any possible serious accident related to the product must be reported to Lascod and to the Competent Authority of the Member State in which the user is established.

Net weight: 450 g.

Net volume after mixing: 1250 ml.

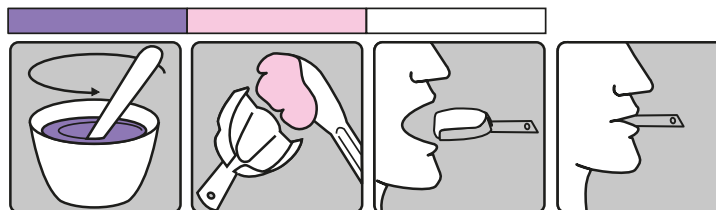(*) Measurements taken at 23°C with deionised water.

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