All Cleaner powder

IT

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All Cleaner powder

EN

Pour the envelope's content (50g.) in one liter of water. The slight developed effervescence helps the process of dissolution. If required mix it until the powder is completely dissolved. **It is suggested to clean the impression trays by any adhesive residues before immersion into All Cleaner solution.** For alginate, plaster or cement residual parts, which are normally deposited on impression trays and instruments, a ready to use All Cleaner solution approximately requires, on first use, the following timings of immersion: alginate 3h (0,05g/h); plaster 6h (0,04g/h); cement 8h (0,03g/h). The sensible increasement of the mentioned timing indicates the necessity to replace the solution. An extended treatment causes deterioration of aluminium impression trays and instruments. After immersion rinse abundantly with water. Before using instruments and impression trays clean and sterilize them. **It can be used with ultrasonic cleaners.**

H319 - Causes serious eye irritation;
H335 - May cause respiratory irritation;
EUH208 - Contains PEPPERMINT EXTRACT. May produce an allergic reaction;
P261 - Avoid breathing dust/fume/gas/mist/vapours/spray;
P280 - Wear eyes protection/ face protection;
P312 - Call a POISON CENTER /doctor if you feel unwell;
P403+P233 - Store in a well-ventilated place. Keep container tightly closed;
Contains: CITRIC ACID MONOHYDRATE

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CONCENTRATED

EN

Dilute 1 volume of solution to 9 volumes of water. **It is suggested to clean the impression trays by any adhesive residues before immersion into All Cleaner solution.** For alginate, plaster or cement residual parts, which are normally deposited on impression trays and instruments, a ready to use All Cleaner solution approximately requires, on first use, the following timings of immersion: alginate 3h (0,05g/h); plaster 6h (0,04g/h); cement 8h (0,03g/h). The sensible increase of the mentioned timing indicates the necessity to replace the solution. An extended treatment causes deterioration of aluminium impression trays and instruments. After immersion rinse abundantly with water. Before using instruments and impression trays clean and sterilize them. **It can be used with ultrasonic cleaners.**

25°C
77°F
max. Store in a cool place.