LASCOD

Fillfix

non eugenol

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FILLFIX

Zinc oxide-based dental cement, without eugenol, for temporary cementation of crowns, bridges, partial crowns, pins, inlays and onlays. Compliant with ISO 3107 - Type 1 standard. For professional use.

Mixing ratio	1:1 vol. Base : Catalyst = 2,0 g : 1,0 g
Mixing time*	40'' - 60''
Working time* (after mixing time)	1' - 3'
Setting time**	3' - 6'

* Measurements taken at 23 +/- 2°C with a relative humidity of 50 +/- 5%.

** Measurement taken at 37 +/- 1°C with a relative humidity of 95 +/- 5%.

Variations in temperature and humidity can affect the measured times.

INSTRUCTIONS FOR USE:

Extrude a quantity of the same length of base paste and catalyst on the supplied mixing pad, or, alternatively, on a plate made of plastic or glass.

Immediately close each tube with its cap after the extrusion of each component. Be careful not to invert the caps.

Vigorously mix the two components for the recommended time, preferably using steel or plastic spatulas, until the preparation becomes smooth and homogeneously colored.

Dry the prepared tooth and the surface of the restoration. Apply a thin layer of mixed cement on the inner surfaces of the restoration using a spatula or a disposable brush. Place the restoration by applying a slight pressure. Make sure that the cement is hardened and remove the excess material.

Fillfix doesn't leave sticky residues on the restoration, it is easily removed without excessive loss of time.

WARNINGS:

Wear gloves and protective clothing. Avoid contact with eyes. In case of contact with eyes wash thoroughly with water. Use for temporary fixation not exceeding 30 days Use within the expiry date and for the applications described. The product is cytotoxic. Do not apply on reduced dentin thicknesses (less than 1.5 mm). Store in a cool and dry place.

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