CE

LASCOD

Chenesyl

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APPLICATIONS

Silicone with ideal viscosity for monophase technique, it can be used also with Super Light Body / Light Body / Regular Body wash silicones as support material inside individual impression tray or standard stainless steel impression tray.

PREPARATION

Dry the areas to be reproduced and prevent further inflow of liquid into them.

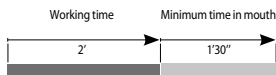
Instructions for dispenser 50ml (1:1):

Load the cartridge into the dispenser (1) and after removing the cap, extrude a small amount of silicone from the two nozzles, moving the lever several times (2). Insert the mixing tip into the cartridge and if necessary, place the small oral tip on this for perfect and precise gingival extrusion, press the lever to extrude the Ghenesyl Mono from the mixing tip. Load the mixed material into the impression tray (3) and insert it into the mouth (4).

Instructions for dispenser 380ml (5:1):

Remove the protective cap of the cartridge (1) and insert it into the automatic mixer. Extrude a small amount of silicone, clean and remove any excess of material (2) Insert the mixing tip into the cartridge (3) and lock it by the fixing ring (4). Start the machine following the manufacturer's instructions for use. Then proceed as indicated at points (3) and (4) of 50 ml dispenser version.

It is important to check that the extruded silicone is homogeneously coloured. To preserve correctly any product left inside the cartridge, leave the mixing tip fixed until the next use. Before applying a new mixing tip, ensure that the silicone comes out of the nozzles regularly. If the cartridge remains unused for a long period, it may become blocked by hardened silicone, which must be removed with a thin instrument before a new mixing tip is placed on.

TECHNICAL DATA	
ISO 4823 classification	Type 2
Mixing ratio	1:1 (50ml) - 5:1 (380ml)
Mixing time	(automixing)
Working time *	2'
Minimum time in mouth	1'30"
Recovery from deformation	99,9 %
Strain in compression	2,5 %
Linear dimensional change at 24 hours	- 0,02 %
Detail reproduction	5µm
Hardness Shore A	60

* Measurement at 23 +/- 2°C and 50 +/- 10% relative humidity.

WARNINGS

The above times can be influenced by:

- variations in the temperature of the material and/or environment (higher temperatures reduce them while lower temperatures prolong them);
- humidity;

Avoid direct or indirect contact with latex rubber dams, gloves or with components of such material. Contact with latex inhibits the hardening of polyvinylsiloxanes. Accurately wash your hands and use vinyl/nitrile gloves.

To eliminate traces of saliva or blood after taking the impression, rinse under running water and dry well.

The impression can be stored without any special precautions. Cast the model in plaster at least 30 minutes after removal from the mouth. Optimal results can be obtained with Ortotipo, Kromotipo or Singletipo class III or IV plaster from the Lascod line.

For disinfection, immerse the impression for 5 minutes in a solution of glutaraldehyde (concentration 0.5%-2%) or sodium hypochlorite (concentration 0.1%-1%). Rinse the impression under running water after disinfection.

IMPRESSION TRAY

All types of rigid impression trays can be used. Use of the Lascod Exploit impression tray is recommended.

OTHER AVAILABLE SILICONES

For dental clinic: condensation silicone SILAXIL, addition silicones OKLUREST and GHENESYL Putty (Hard and Soft); Ghenesyl Body (Superlight, Light, Regular and Heavy).

For laboratory: condensation silicone ERGASIL; addition silicones ERGAMIX and XILGUM.

GUARANTEE

LASCOD SpA guarantees the quality of its products and undertakes to replace them whenever defects in production are found within the expiry dates. LASCOD SpA will not be held responsible and will not recognize any damage that is direct, indirect, accidental or caused by incorrect use of the material. Check the suitability of the product prior to use.

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