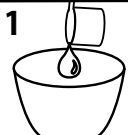
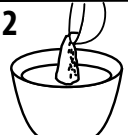
tropical alginate

		TYPE I fast	TYPE II normal
			
	20 ml. + 10,5 g. 		
	40 ml. + 21 g. 		
1 	2 	60" MIXING TIME 	60" MIXING TIME 
		1'50" WORKING TIME including mixing time 	2'50" WORKING TIME including mixing time 
		2'00" INITIAL SETTING TIME including working time 	3'00" INITIAL SETTING TIME including working time 
			
		60" in mouth	

Measurements taken at 23 °C with deionised water.

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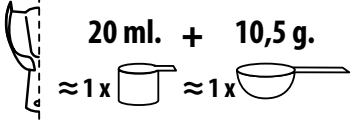
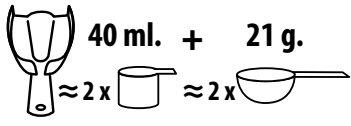
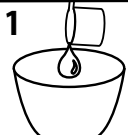
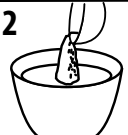
tropical alginate

		TYPE I fast	TYPE II normal
 20 ml. + 10,5 g. 		60" MIXING TIME	60" MIXING TIME
 40 ml. + 21 g. 		1'50" WORKING TIME including mixing time	2'50" WORKING TIME including mixing time
1  2  		2'00" INITIAL SETTING TIME including working time	3'00" INITIAL SETTING TIME including working time
		 60" in mouth	

Measurements taken at 23 °C with deionised water.

TYPE I: FAST SETTING - TYPE II: NORMAL SETTING

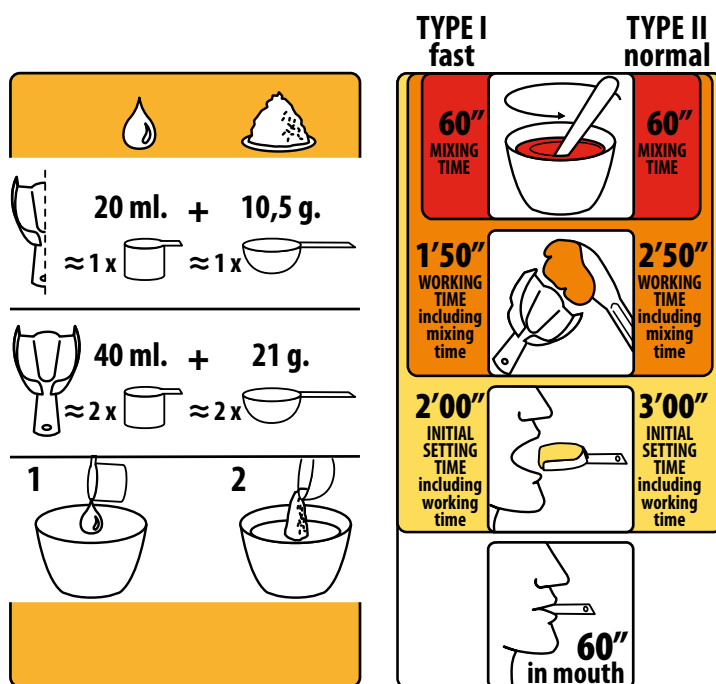
It does not require special treatment before use. Follow the powder/water ratio. Pour the powder into the water, mix vigorously. Work in compliance with the declared times (*), measurements were taken at 23°C with deionized water. When using specific mechanical mixers for alginate-based impression materials, follow the manufacturer's instructions. Variations in powder/water ratio, in temperature and/or water hardness and in the speed and/or friction during mixing cause changes in the working time and in the initial setting time. **The colour changes indicate the different work phases (see table).** Wash, rinse, shake, disinfect the impression (specific alcohol-based disinfectants are recommended), then pour the model. Recommended plasters: Kromotypo 3, Singletypo 3, Kromotypo 4, Singletypo 4, Ortotypo. **WARNINGS:** or use, wear face mask, protective gloves, eye protection and protective clothing. The powder is "Dust Free", however, caution is recommended to avoid inhalation.

In case of contact with eyes wash thoroughly with water. After taking the powder close the bag immediately. Store in a cool and dry place. Use within the expiry date. Do not use on patients allergic to mango flavour. Any possible serious accident related to the product must be reported to Lascod and to the Competent Authority of the Member State in which the user is established.

Net weight: 450 g. - Net volume after mixing: 1250 ml.

tropical alginate

Measurements taken at 23 °C with deionised water.

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tropical alginate

		TYPE I fast	TYPE II normal
20 ml. + 10,5 g. 		60" MIXING TIME	60" MIXING TIME
40 ml. + 21 g. 		1'50" WORKING TIME including mixing time	2'50" WORKING TIME including mixing time
1 	2 	2'00" INITIAL SETTING TIME including working time	3'00" INITIAL SETTING TIME including working time
		60" in mouth	

Measurements taken at 23 °C with deionised water.

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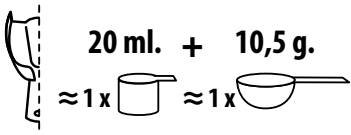
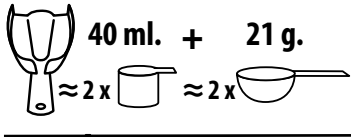
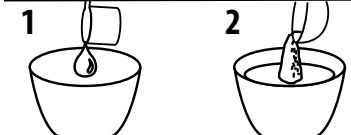
tropical alginate

		TYPE I fast	TYPE II normal
			
20 ml. + 10,5 g. 		60" MIXING TIME	60" MIXING TIME
40 ml. + 21 g. 		1'50" WORKING TIME including mixing time	2'50" WORKING TIME including mixing time
1	2	2'00" INITIAL SETTING TIME including working time	3'00" INITIAL SETTING TIME including working time
			
			

Measurements taken at 23 °C with deionised water.

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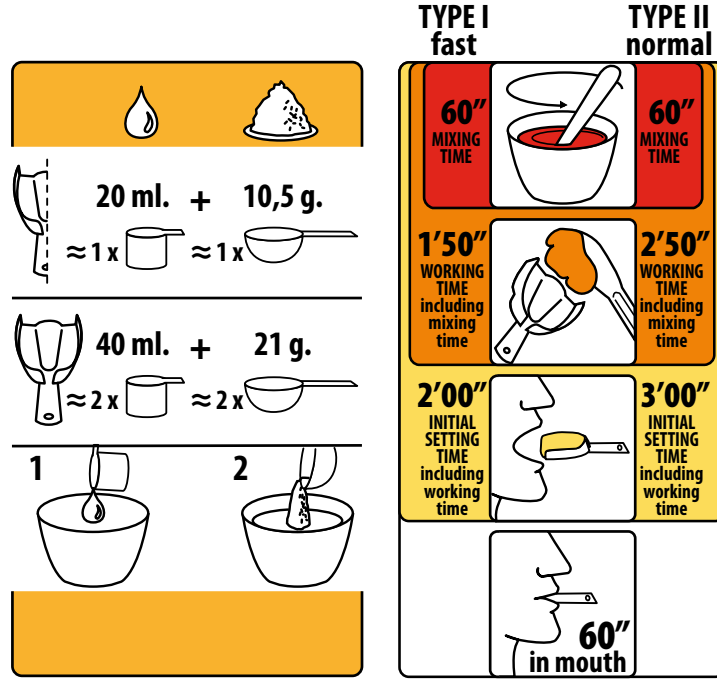
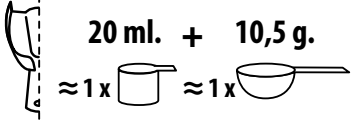
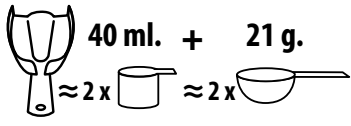
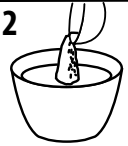
tropical alginate

		TYPE I fast	TYPE II normal
			
20 ml. + 10,5 g. 		60" MIXING TIME	60" MIXING TIME
40 ml. + 21 g. 		1'50" WORKING TIME including mixing time	2'50" WORKING TIME including mixing time
1  2 		2'00" INITIAL SETTING TIME including working time	3'00" INITIAL SETTING TIME including working time
			

Measurements taken at 23 °C with deionised water.

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