CE

LASCOD

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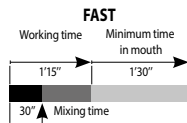
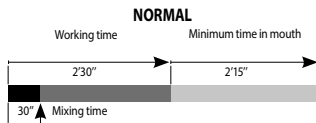
APPLICATIONS - Preliminary impressions for the two-step putty/wash and base for the sandwich technique.
The impression can be electroplated.

TECHNICAL DATA	PUTTY SOFT NORMAL	PUTTY SOFT FAST	PUTTY HARD NORMAL	PUTTY HARD FAST
ISO 4823 classification	Type 0		Type 0	
Mixing time	30"			
Working time *	2' 30"	1' 15"	2' 30"	1' 15"
Minimum time in mouth	2' 15"	1' 30"	2' 15"	1' 30"
Recovery from deformation	99,9%		99,9%	
Strain in compression	4%		1,9%	
Linear dimensional change at 24 hours	- 0,02%		- 0,02%	
Detail reproduction	25µm		25µm	
Hardness Shore A	50		63	

*Measurement at 23 +/- 2°C and 50 +/- 10% relative humidity.

The above times can be influenced by:

- variations in the temperature of the material, the environment and/or the operator's hands during mixing (higher temperatures reduce them while lower temperatures prolong them)
- changes in the speed and/or force employed during mixing
- humidity.

INSTRUCTIONS FOR USE - Dry the areas to be reproduced and prevent further inflow of liquid into them. Measure out the two pastes in a ratio of 1:1, using the corresponding coloured scoops (1). Mix the two components vigorously to obtain a homogeneously coloured mass (2). Place the mixed mass on the impression tray (3) and insert it into the mouth (4). All types of rigid impression trays can be used. Use of the Lascod Exploit impression tray is recommended. Do not invert the lids of the containers and use the corresponding coloured scoop for each paste. Close the containers well after use. To eliminate traces of saliva or blood and guarantee optimal adhesion of the wash impression to the preliminary impression, rinse under running water and dry well.

The impression can be stored without any special precautions. Cast the model in plaster at least 30 minutes after removal from the mouth. For disinfection, immerse the impression for 5 minutes in a solution of glutaraldehyde (concentration 0.5%-2%) or sodium hypochlorite (concentration 0.1%-1%). Rinse the impression under running water after disinfection. Optimal results can be obtained with Ortotipo, Kromotipo or Singletipo class III or IV plaster from the Lascod line.

WARNINGS - Wear gloves and protective clothing. **Use only vinyl/nitrile gloves. Avoid direct or indirect contact with latex rubber dams, gloves or with components of such material. In case you have used latex gloves, wash your hands thoroughly. Contact with latex inhibits the hardening of polyvinylsiloxanes.** Do not use on patients allergic to mint flavour. Store in a cool and dry place. Use within the expiry date and for the applications described. Any possible serious accident related to the product must be reported to Lascod and to the Competent Authority of the Member State in which the user is established.

OTHER AVAILABLE SILICONES - For dental clinics: SILAXIL condensation silicone; GHENESYL and OKLUREST addition silicones.
For laboratories: ERGASIL condensation silicone; ERGAMIX and XILGUM addition silicones.

GUARANTEE - LASCOD SpA guarantees the quality of its products and undertakes to replace them whenever defects in production are found within the expiry dates. LASCOD SpA will not be held responsible and will not recognize any damage that is direct, indirect, accidental or caused by incorrect use of the material. Check the suitability of the product prior to use.

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