CAD
SYSTEM

LASCOD

OKUrest

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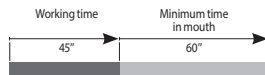
APPLICATIONS

Oklurest is an addition silicone specifically intended for orthodontic occlusion registration, registration keys for gnathological registrations, inter-maxillary registration keys for centered positions, eruptions and ectopic eruptions, registration for cephalometric analysis which also require subsequent scanning with CAD systems.

CHARACTERISTICS

- Scannable using optical/laser/tactile scanning systems
- Maximum control after positioning
- Thixotropic
- Viscosity imperceptible for the patient
- High details definition
- Fast setting time
- Easy repositioning in the mouth and on the models after hardening
- Can be finished using burs

TECHNICAL DATA

Base paste/catalyser ratio	1:1
Working time*	45"
Minimum time in mouth	60"
Detail reproduction	5µm
Linear dimensional change at 24 hours	-0.02%
Hardness Shore A	95

*Measurement at 23 +/- 2°C and 50 +/- 10% relative humidity.

The above times can be influenced by:

- variations in the temperature of the material and/or environment (higher temperatures reduce them while lower temperatures prolong them);
- humidity.

INSTRUCTIONS FOR USE

"Load the cartridge into the dispenser (1) and after removing the cap, extrude a small amount of silicone from the two nozzles, moving the lever several times (2). Insert the mixing tip into the cartridge. Actuate the lever forcefully so as to extrude the silicone from the mixing tip. Apply Oklurest directly and without interruption onto the occlusal surface of the teeth (3). The operator then asks the patient to bite down with maximum intercuspation (4); since it is imperceptible, Oklurest facilitates the closure in a natural manner. Wait for complete hardening and remove the obtained impression.

The thixotropic consistency of Oklurest facilitates its control and thus prevents the material from entering the undercut areas and from dripping during positioning on the dental arches independently of the method used (occlusal or vestibular).

To preserve any product remaining in the cartridge correctly, leave the mixing tip fixed until the next use. Before applying a new mixing tip, ensure that the silicone comes out of the nozzles regularly. If the cartridge remains unused for a long period, it may become blocked by hardened silicone, which must be removed with a thin instrument before a new mixing tip is placed on.

To eliminate traces of saliva or blood after taking the impression, rinse under running water and dry well.

For disinfection, immerse the impression for 5 minutes in a solution of glutaraldehyde (concentration 0.5%-2%) or sodium hypochlorite (concentration 0.1%-1%). Rinse the impression under running water after disinfection.

Store the impression at room temperature without particular precautions other than protection from mechanical stress.

WARNINGS

Wear gloves and protective clothing. **Use only vinyl/nitrile gloves. Avoid direct or indirect contact with latex rubber gloves or with components of such material. In case you have used latex gloves, wash your hands thoroughly. Contact with latex inhibits the hardening of polyvinylsiloxanes.** Store in a cool and dry place. Use within the expiry date and for the applications described.

Any possible serious accident related to the product must be reported to Lascod and to the Competent Authority of the Member State in which the user is established.

OTHER AVAILABLE SILICONES

For dental clinics: SILAXIL condensation silicone; KROMOPAN SIL and GHENESYL addition silicones.

For laboratories: ERGASIL condensation silicone; ERGAMIX and XILGUM addition silicones.

GUARANTEE

LASCOD SpA guarantees the quality of its products and undertakes to replace them whenever defects in production are found within the expiry dates. LASCOD SpA will not be held responsible and will not recognize any damage that is direct, indirect, accidental or caused by incorrect use of the material. Check the suitability of the product prior to use.

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